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## Use instruction COVID-19 FFP2 mask 'Corona Protection Mask, n°434050\_4'

Revision 2.2

**This filtering half mask is manufactured for COVID-protection only. This filtering half mask is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19.**

These Corona Protection Masks are certified by BSI (Nederlands), Notified body n°2797 (Say Building, John M. Keynesplein 9, 1066 EP Amsterdam), to technical specification Annex II (EHSR) of the PPE Regulation (EU) 2016/425 and BSI's PPE for Healthcare Professionals 2020/403 – RPE Technical Specification. For the declaration of conformity follow this link: <https://www.auping.com/declarationofconformity-ffp2mask>

### Pay attention:

- The choice of the correct mask, the correct use, the correct training for the use and the correct storage of the mask are essential to help protect the user against the corona virus. Failure to do so in whole or in part may adversely affect the health of the wearer or even lead to serious life-threatening illness or disability.
- The mask is a single-use product: only used for a maximum period of one shift. For any contact with a patient suffering from the coronavirus, the health care provider is advised to use a new corona protection mask.
- It is only suitable for (and tested with) wearers without a beard. Beards under the seal reduce the seal / protection.
- Before use, first check that the storage period has not been exceeded.
- Before use, check that the mask is undamaged.
- Leave the mask in the original packaging until use.

### Use instructions for the Corona Protection Masker (see figures above)

1. Wash your hands
2. Remove the mouth mask from the packaging
3. Hold the mask over the nose and mouth with one hand and pull with the other the elastic bands over the head, one above the ears and one below.
4. Check that the mask is properly positioned. If necessary, stretch the elastic bands a bit for better comfort.
5. Place your fingers on the nose and press the nose clip with both hands so that it follows the shape of the nose and seals properly.
6. Place both hands on the mask and exhale firmly and check whether the air escapes past the mask. If that is the case, reposition the mask and try again. In case air keeps escaping past the mask, the mask will not fit your face properly and you cannot use it.
7. Leave the contaminated area immediately when
  - a. Breathing becomes difficult
  - b. Nausea or other discomfort occurs
8. Take off mask after use:
  - a. Disinfect your hands
  - b. do not touch the blue part, but use the bands to take off the mask
  - c. Disinfect your hands again
  - d. Dispose of the contaminated mask in the appropriate waste in accordance with national guidelines.